HEINE Standard F.O. L Laryngoskop Ladegriff

1. 2. 3. 4.

CE 93/42/EWG/CEE/EEC

HEINE

HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49(0)8152/38-0 · Fax +49(0)8152/38-202
E-mail: info@heine.com · www.heine.com

med 1109 1/10.09

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ISO 7376

(Anaesthetic and respiratory equipment – Laryngoscopes for tracheal intubation)

HEINE Standard F.O. L Laryngoscope Rechargeable Handle

1. 2. 3. 4.

CE 93/42/EWG/CEE/EEC

Instructions

Read these instructions carefully before using the HEINE Standard F.O. L Laryngoscope Rechargeable Handle and keep them in a safe place for future reference.

In accordance with local regulations this product should be disposed of as an electronic device separately.

Safety Information

Range of applications: HEINE Standard F.O. L Laryngoscope rechargeable handles are designed for use with F.O. laryngoscopes for intubation only. The performance of the handles can only be guaranteed if genuine HEINE rechargeable batteries (Li-ion L, see separate instruction leaflet for HEINE rechargeable batteries) and other spare parts are used.

General safety:

- Do not open the bottom insert.
- All repairs other than a change of batteries should be carried out by qualified persons.

Safety in use:

- Use only blades and handles with a green mark together.
- Do not use in the presence of strong magnetic fields.
- Charge rechargeable handles before use.
- The handles should not be used in the presence of explosive substances.
- Rechargeable handles should not be used while they are being recharged in the charger.

Application

1. HEINE Standard F.O. L Laryngoscope Rechargeable Handle

HEINE Standard F.O. L Laryngoscope rechargeable handles consist of four basic components:

- Handle shell (1)
- Battery insert (2)
- Li-ion L rechargeable battery (3)
- Bottom insert (4)

Changing the rechargeable battery:

- Unscrew the battery insert from the handle shell and remove it.
- Unscrew the bottom insert from the battery insert and remove it complete with the rechargeable battery.
- Pull the rechargeable battery out of the bottom insert.
- Insert the new rechargeable battery into the bottom insert with an audible „click“.
- Screw the bottom insert complete with the rechargeable battery back into the battery insert.
- Screw the battery insert back into the handle shell.

Changing the bulb:

- Unscrew the battery insert from the handle shell and remove it.
- Unscrew the bulb from the battery insert.
- Replace the new bulb into the battery insert carefully and screw it in.
- Screw the battery insert back into the handle shell.

2. Angled F.O. L handle

The angled F.O. L handle consists of the following components:

- Handle head
- Outer sleeve
- Set of batteries or rechargeable battery
- Bottom insert

Changing the rechargeable battery:

- Unscrew the bottom insert from the outer sleeve, remove the battery.
- Insert the new battery into the outer sleeve.
- Screw the bottom insert back into the outer sleeve.

Changing the bulb:

- Unscrew the handle head.
- Unscrew the nut with the key supplied. Alternatively, use a coin inserted into the slits in the nut and the surrounding part and press down.
- Unscrew the bulb completely and remove the bulbholder and bulb.
- Place the new bulb in the opening and place the bulbholder over it.
- Screw in the nut.
- Fit the handle head so that both marks coincide and screw the handle head firmly in.

The rechargeable battery

This handle has been developed specifically for use with the HEINE Li-ion L battery (see the instruction leaflet HEINE Rechargeables batteries) and the NT300 charger (for details see the instruction leaflet NT300).

The NT200 charger can also be used, but does not offer the same performance. The handle can be modified by exchanging the bottom insert and the rechargeable battery.

The bottom insert incorporates a charge status display which shows the state of charge of the battery. This activates automatically when the handle is removed from the charger or when it is switched on. After 10 seconds the display switches off.

The display gives the following information:

- Orange flashing: < 10%
- Orange: 10 – 33%
- Orange/green: 33 – 66%
- Orange/green/green: 66 – 100%

Cleaning / Disinfection / Sterilising

The following instructions must be followed according to national standards, laws and guidelines. Any such measures must be carried out by qualified personnel. Before cleaning/disinfecting remove the batteries. See the instructions for the appropriate handle type.

For cleaning and disinfection of components, only those products which are approved for use on medical devices with plated surfaces and plastics (handles contain plastic components, e.g. bottom cap) may be used. After treatment, the instrument must be thoroughly rinsed with de-mineralised water to avoid reactions with following preparation steps and material damage. The manufacturer's recommendations for use must be followed.

F.O. L Handle

Cleaning / Disinfection:

The handle sleeve can be cleaned and disinfected manually (wiping / soaking) or by machine (thermal disinfection up to 93 °C). Battery insert can be cleaned and disinfected manually (wiping) only.

Sterilisation:

Steam Sterilisation at 134 °C is allowed for the handle sleeve only.

F.O. L Angled Handle

Cleaning / Disinfection:

Handle sleeve and handle head can be cleaned and disinfected manually (wiping/soaking) or by machine (thermal disinfection up to 93 °C). Battery insert can be cleaned and disinfected manually (wiping) only.

Sterilisation:

Steam Sterilisation at 134 °C is allowed for the handle sleeve and handle head only.

Accessories and spare parts

Standard F.O. L Handle

Battery insert without bottom cap	F-002.22.859
Bottom cap	F-002.22.887
Rechargeable battery	X-007.99.383
Handle shell	F-000.22.824
Spare bulb	X-002.88.044

F.O. L Angled Handle

Bottom cap	X-002.99.396
Rechargeable Battery	X-007.99.383
Spare bulb	X-002.88.044

Applicable standards:

ISO 7376

(Anaesthetic and respiratory equipment – Laryngoscopes for tracheal intubation)

Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries). We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories.

(Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

Bought from _____

date _____